

WINDOWS VISTA

VIDEO OTOSCOPE USER MANUAL



BioCam2.0TM
The new revolution in clinical diagnosis

1 Windows Vista- Installation

How to Install the BioCam2.0™ Software on Vista Page 1


2 Windows Vista- Using The BioCam2.0™

Image Capture Page 8


Video Capture Page 9

Windows Vista-Installation

How to Install the BioCam2.0™ Software on Vista

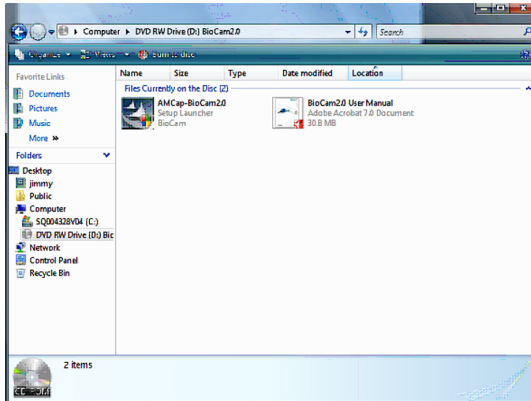

1. Insert installation CD. Double Click on the AMCap-BioCam2.0 setup launcher.

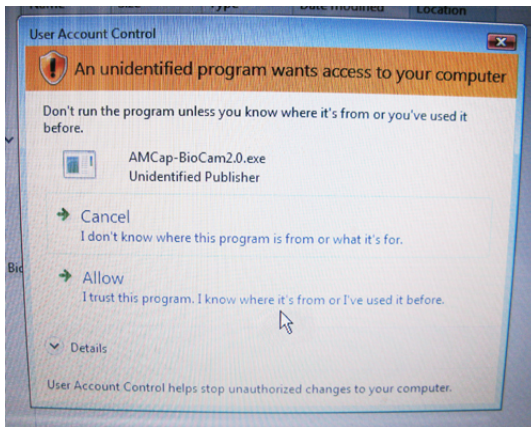


2. The User Account Control will appear. Select Allow to continue with the installation.


1

Windows Vista-Installation

How to Install the BioCam2.0™ Software on Vista

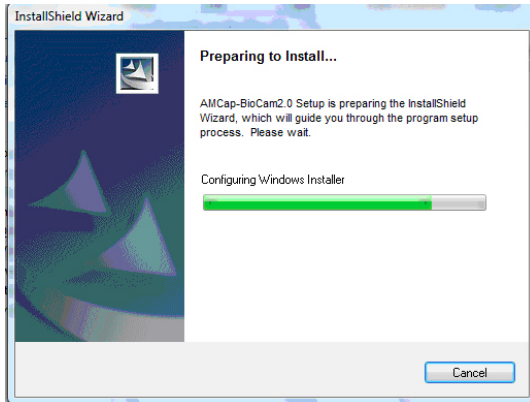

3. Windows will configure and prepare for installation to initiate.

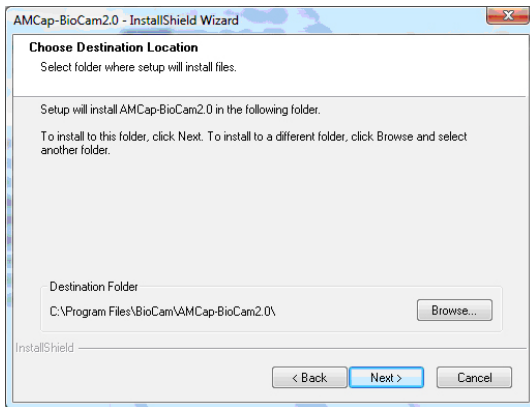


4. When the **InstallShield Wizard** appears, select **Destination folder** and Click **Next** to continue with installation.


1

Windows Vista-Installation

How to Install the BioCam2.0™ Software on Vista

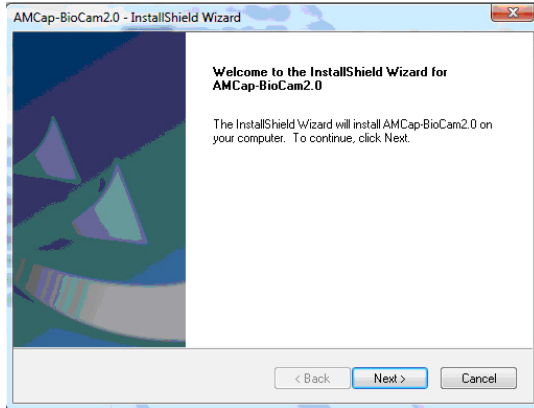


5. Click Next.

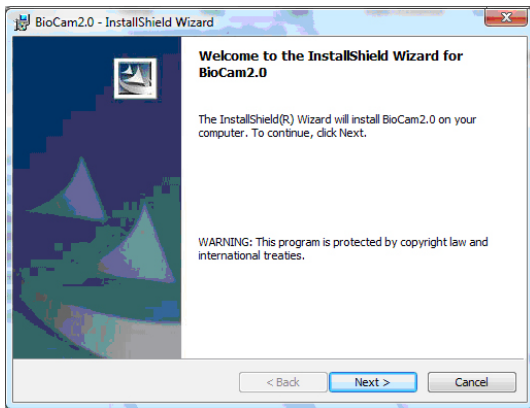


6. Click Next.

Windows Vista-Installation

How to Install the BioCam2.0™ Software on Vista

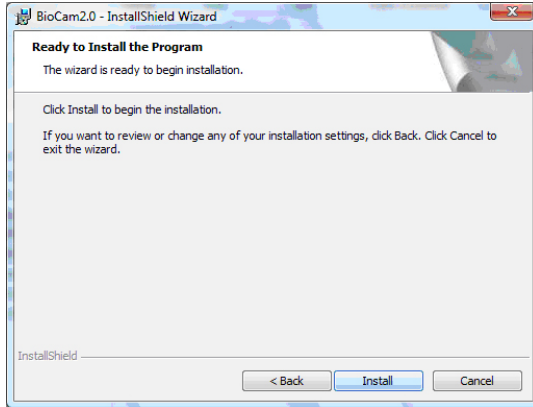

5. Click Install.

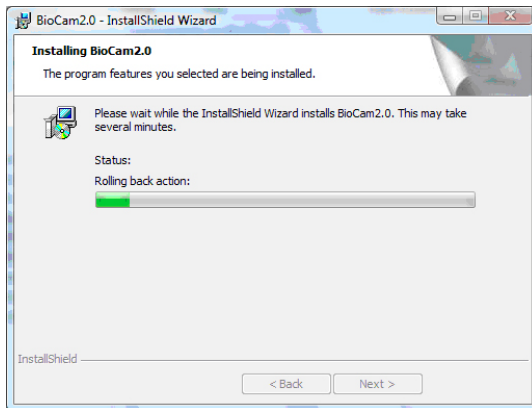


6. Installation will begin.


1

Windows Vista-Installation

How to Install the BioCam2.0™ Software on Vista

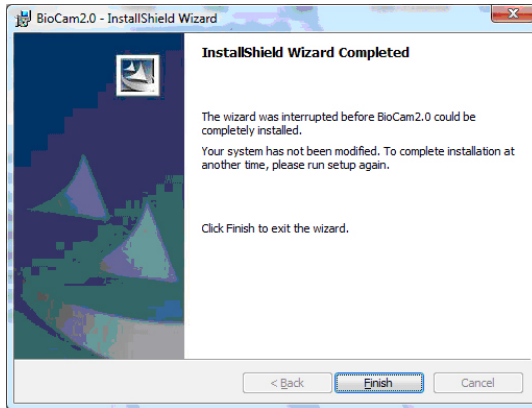

7. Click **Finish** when installation is completed.

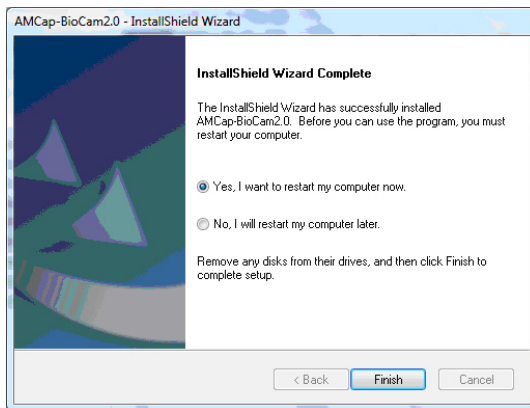

8. Click **Finish** to restart the system. The software is now installed.

1


Windows Vista-Installation

How to Install the BioCam2.0™ Software on Vista


AFTER SYSTEM HAS RESTARTED

9. Once the system has restarted, **plug in** the BioCam2.0 device onto the computer.

* Do not Plug in the BioCam2.0 device before the AMCap BioCam2.0 installation is completed.

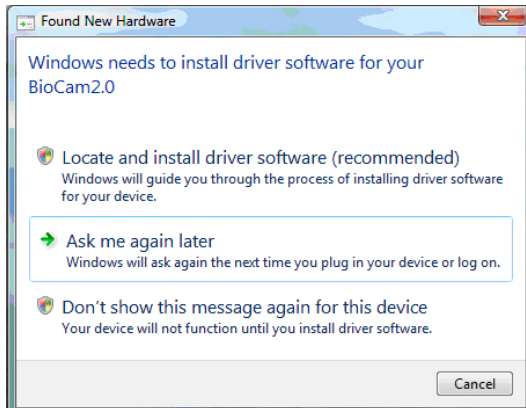

10. The Found New Hardware window will open. Select **Locate and install driver software(Reccomeded)** option to continue.

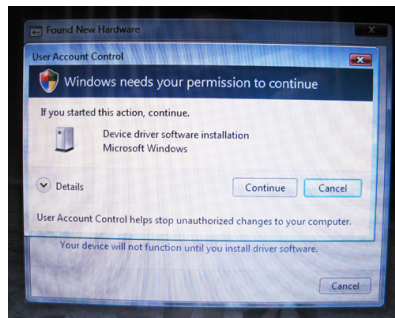


11. Click **Continue** to authorize hardware driver installation.



Windows Vista-Installation

How to Install the BioCam2.0™ Software on Vista

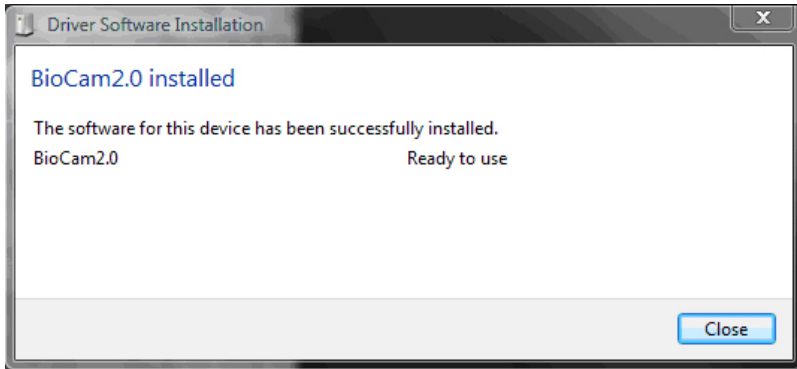


12. Click **Close** when driver software installation is completed. The BioCam2.0™ is ready to be used now.


Windows Vista-Using the BioCam2.0™ Image Capture



1. Press the **Freeze** button on the BioCam2.0™ to capture an image. Image will appear on the AMCap-BioCam2.0 window.

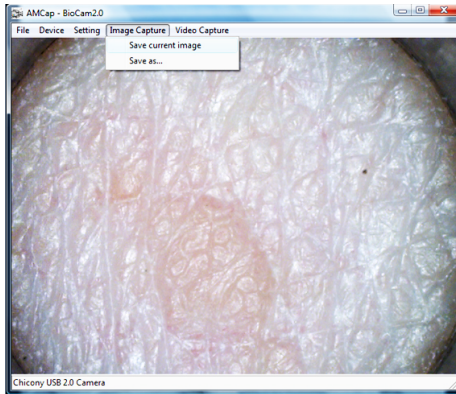


2. Click on the **Image Capture** drop down menu to save the image. Select **Save As**.

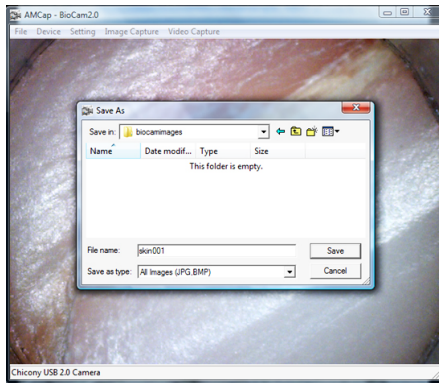


3. Choose **destination folder** and click **Save** to save image. Image can now be viewed anytime in the destination folder.

Windows Vista-Using the BioCam2.0™ Video Capture

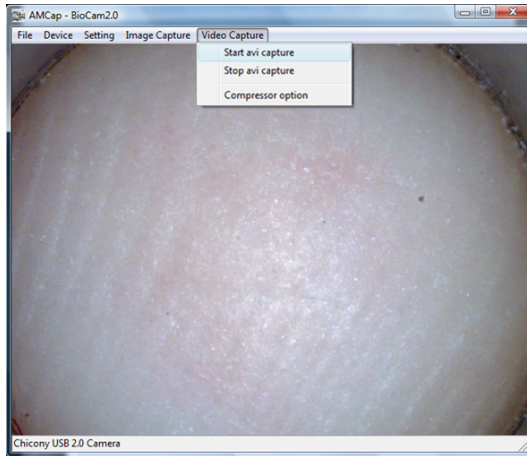


1. Click on the **Video Capture** drop down menu. Select **Start avi capture** to start video recording.

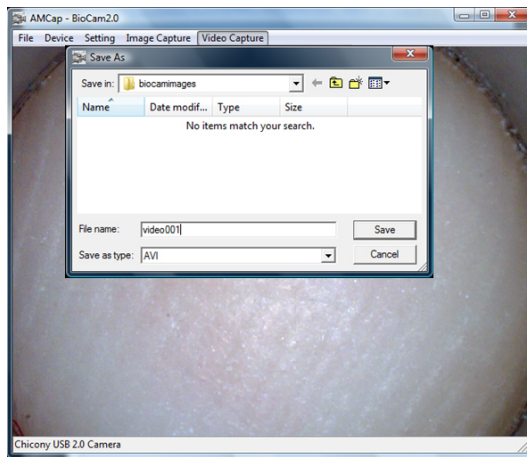

2. Choose **destination folder** and click **Save** to save video. Video capture will start immediately after this.

Windows Vista-Using the BioCam2.0™ Video Capture

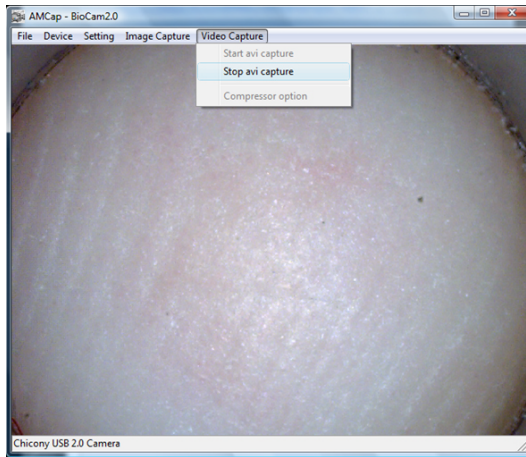

3. To end video recording, click on the **Video Capture** drop down menu. Select **Stop avi capture** to stop recording.