




INSTALLING THE BIOCAM2.0™2.0 ON WINDOWS 7 (32-bit and 64-bit)

1. **Download** The BioCam2.0™2.0 for windows 7 form our website www.goldlineBioCam2.0™.com or www.BioCam2.0™vet.com


The screenshot shows a website page for BioCam 2.0. On the left, there is a navigation menu with links for "Vista 64-bit driver", "Apple Mac driver", "XP & VISTA 32-bit USERS", "AMCap BioCam Installer", "WINDOWS 7 USERS", "Windows 7 32-bit", and "Windows 7 64-bit". Below this is a section for "XP, Vista & Mac Installation Manual" with a "Dear valued customers," message and links to instruction manuals for XP, Vista, and Apple Mac users. The main content area features text about the BioCam 2.0™, including a quote from Lawrence R. Grobman, MD, and a testimonial. A security warning dialog box is overlaid on the page, asking "Do you want to run or save this file?" for the file "AMCap-BioCam2.0WIN764.exe" from "www.goldlinebiocam.com". The dialog box also includes a warning icon and a link to "What's the risk?".

2. **Double Click** on the **installer** to start installation.

3. **User Account Control window** will appear asking "Do you want to allow the following program from an unknown publisher to make changes to this computer?" **Select Yes** to continue with installation. InstallShield Wizard will start to prepare for the AMCap- BioCam2.0 setup.

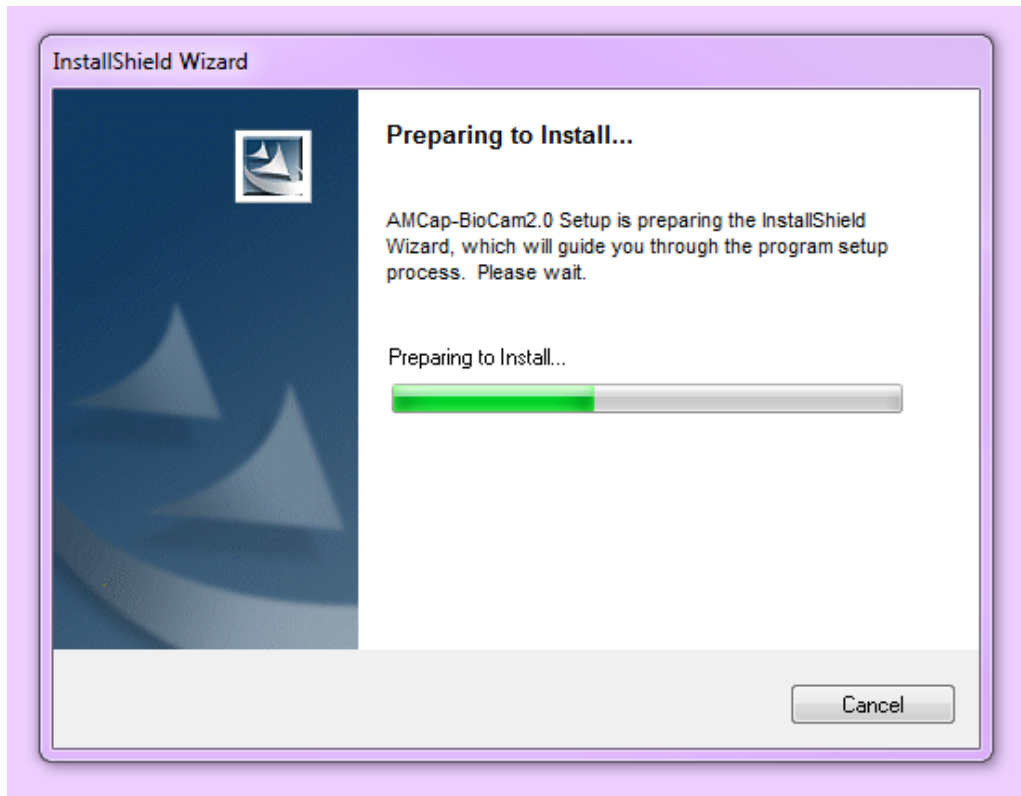


4. AmCap- BioCam2.0™2.0 - Install Shield Wizard will appear . **Select Next.**

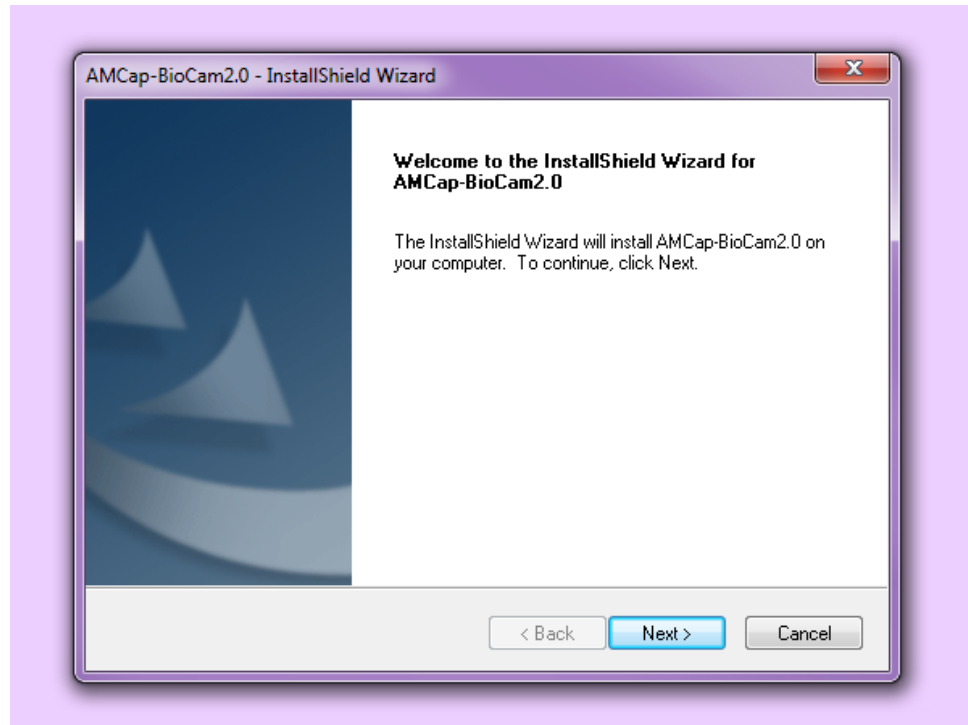





5. Choose destination Location. Destination folder should be in c:\ Program Files (x86)\BioCam2.0™\AMCap-BioCam2.0™2.0\ . **Select Next** to continue with installation.

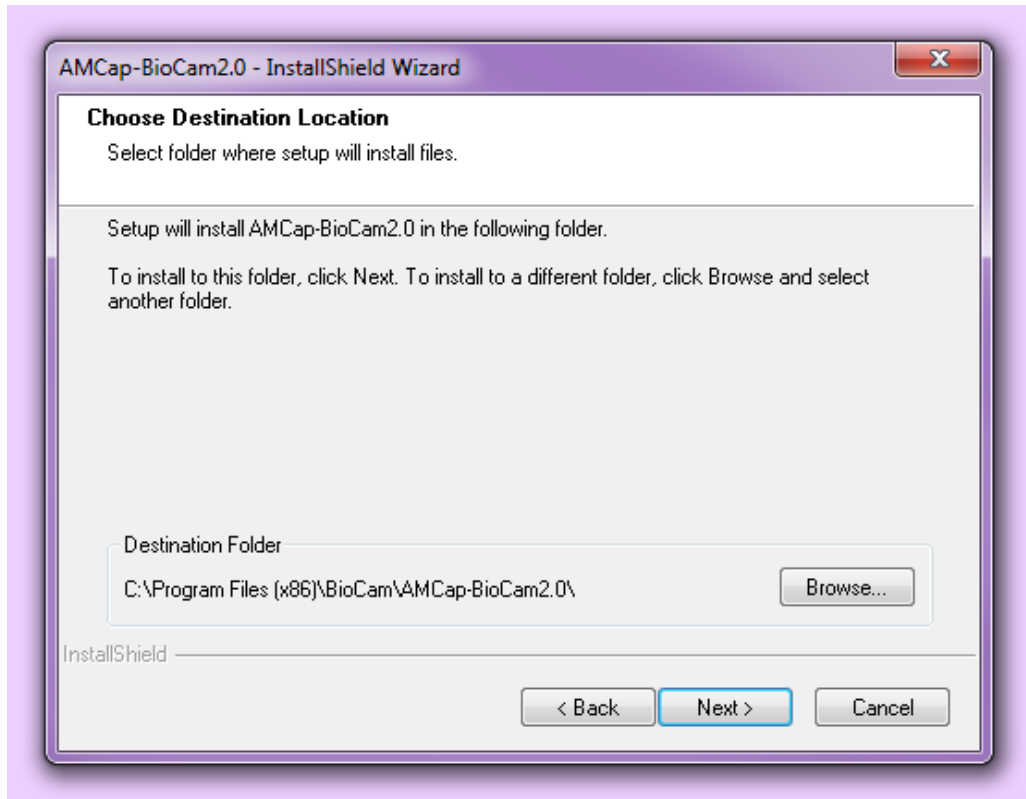


6. **Select Next** to install the Indeo 5 Setup.

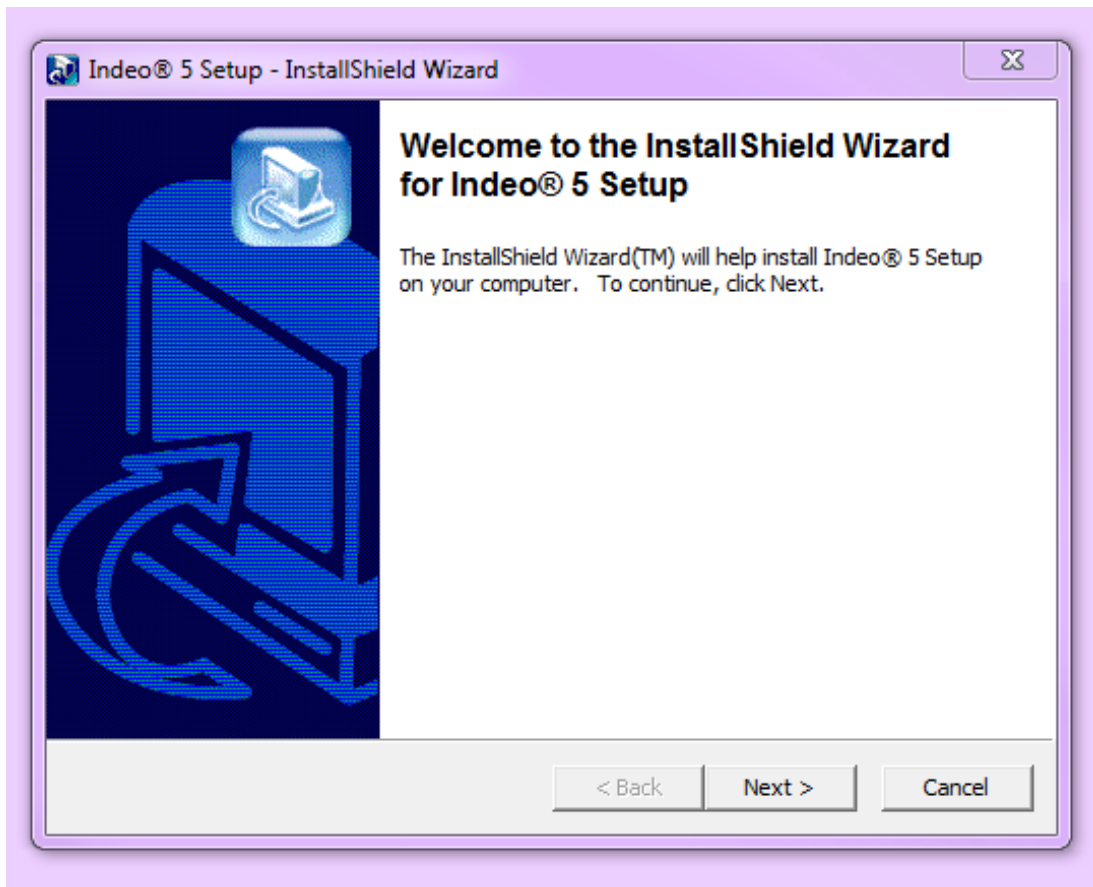


7. Select Next.

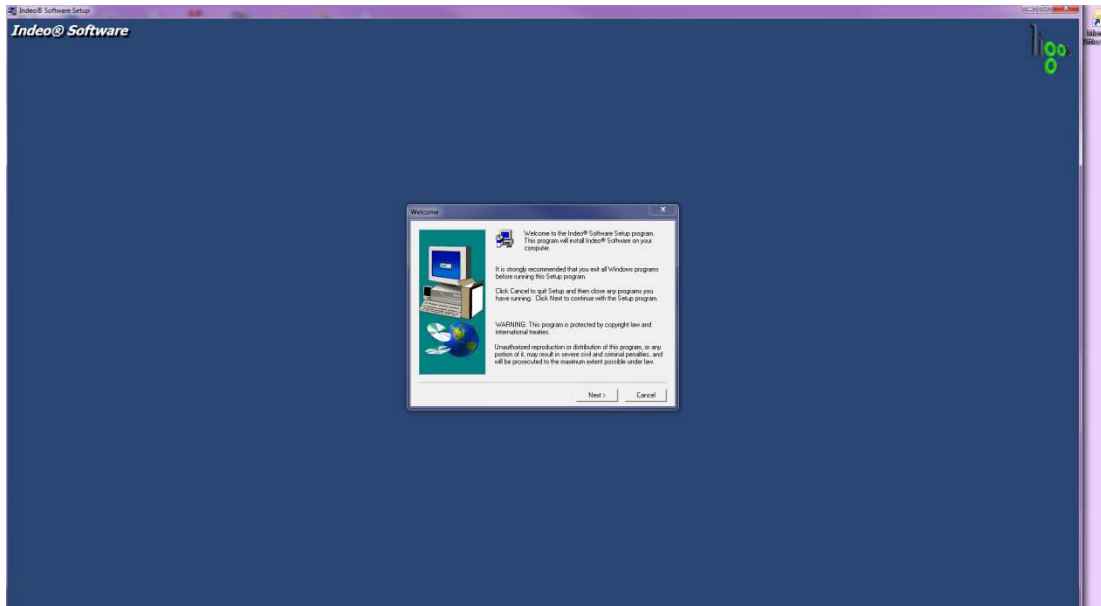


8. Select Yes to agree with the software license agreement.

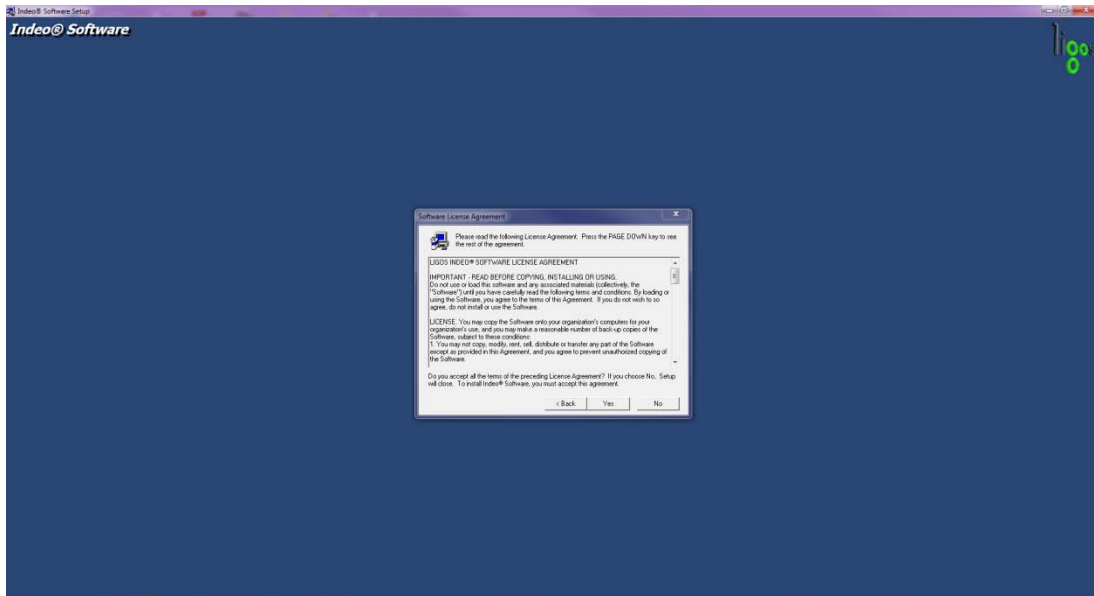


9. Click **Next** once you have selected your destination location (Default location is always the C:\).

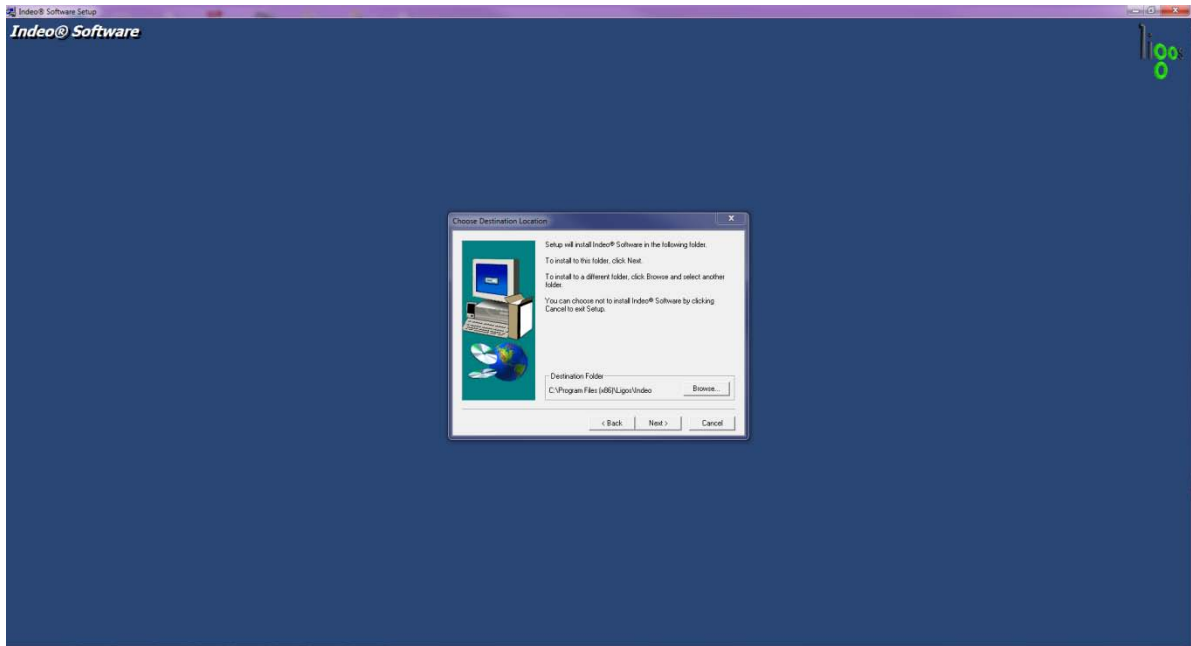


10. Select **Typical** on the Setup type and click **Next**.

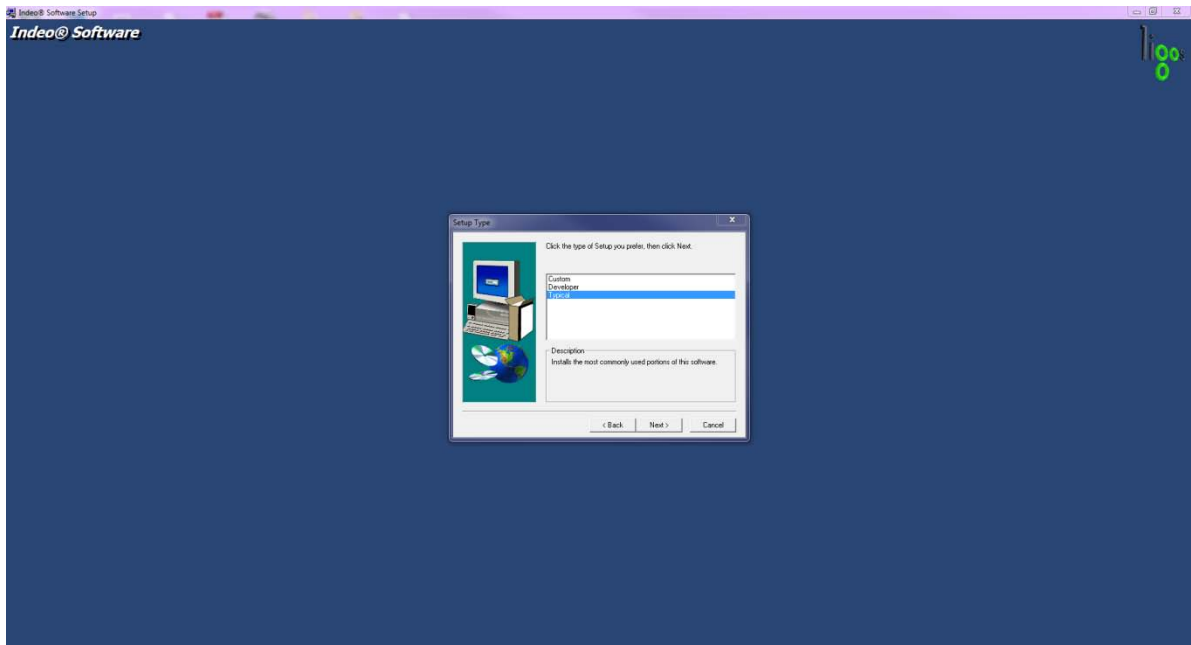


11. **Select Yes** to overwrite Indeo system files.

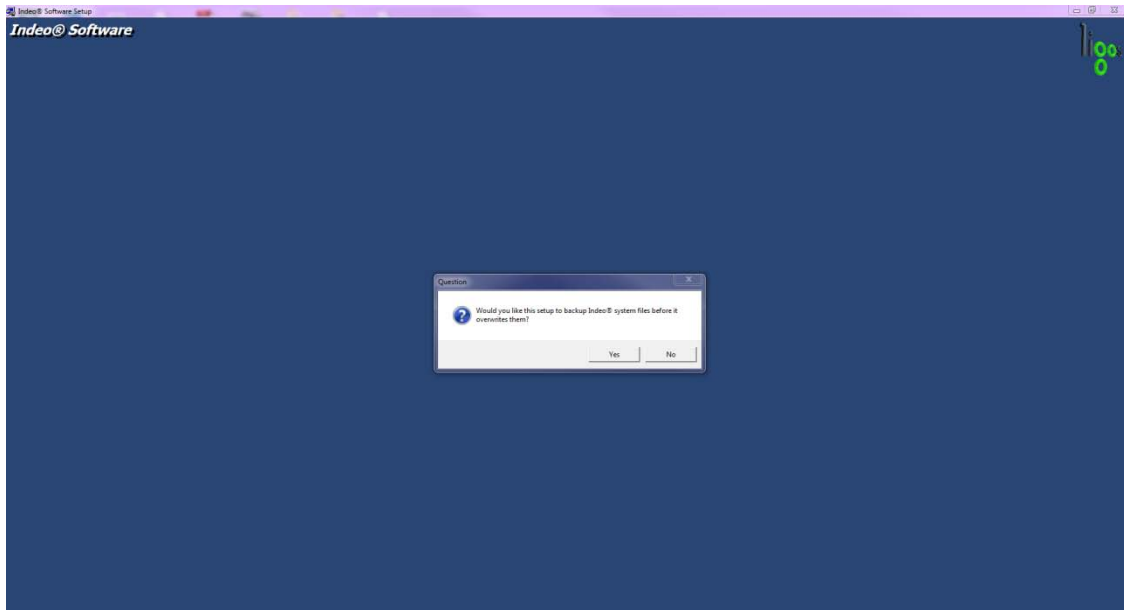


12. **Select Next** to save system file to location C:\

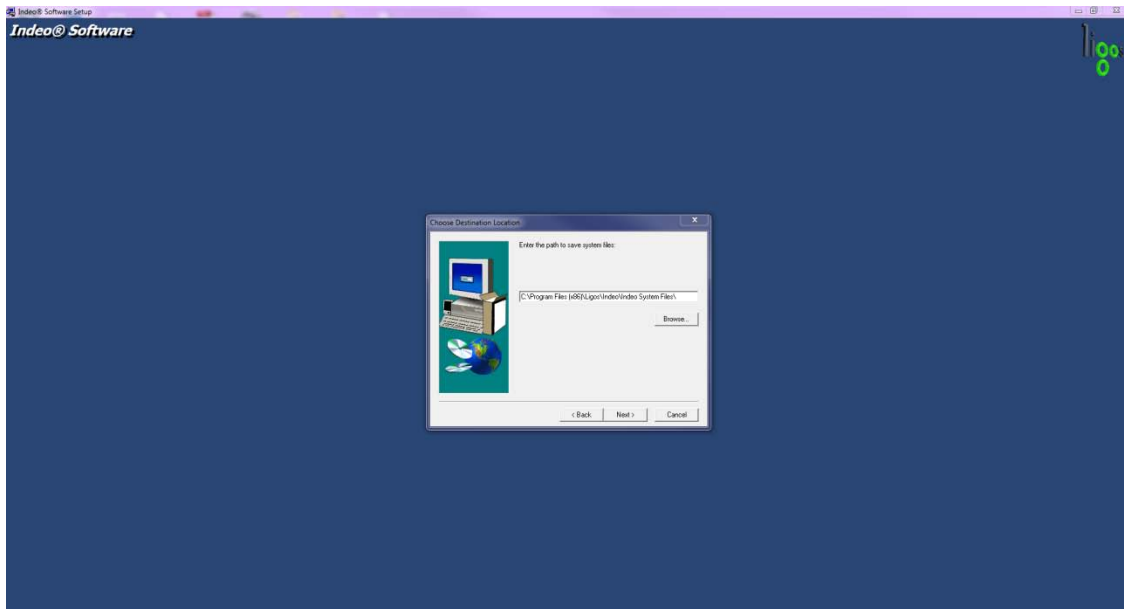


13. **Select Next** to Start copying files.

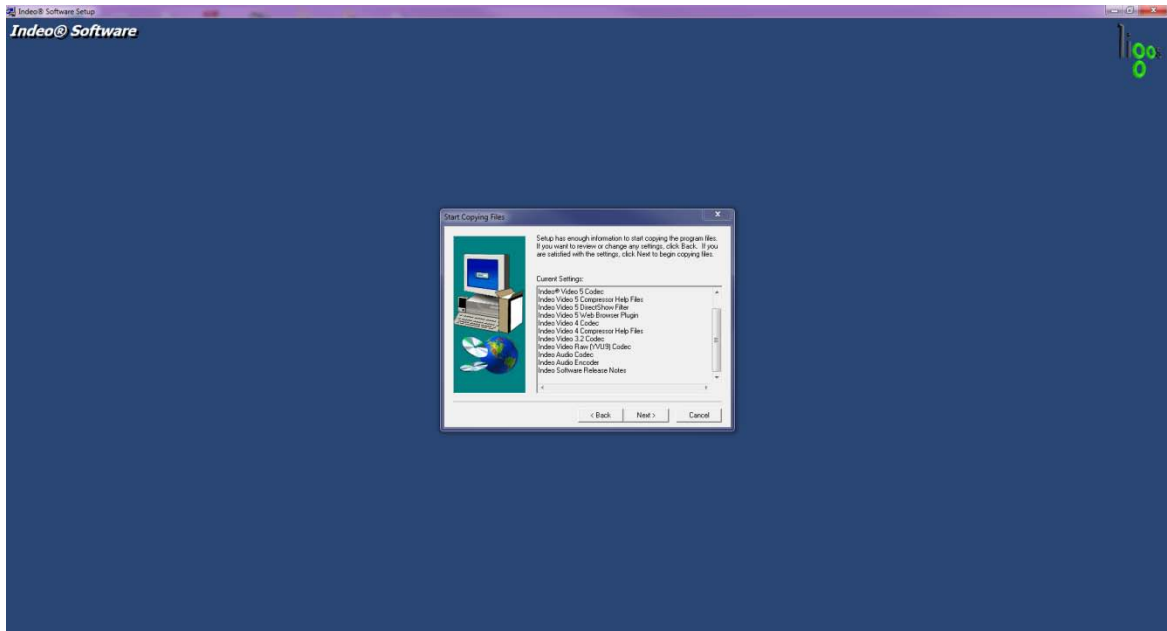


14. **Select Finish** to complete Indeo Software Setup.

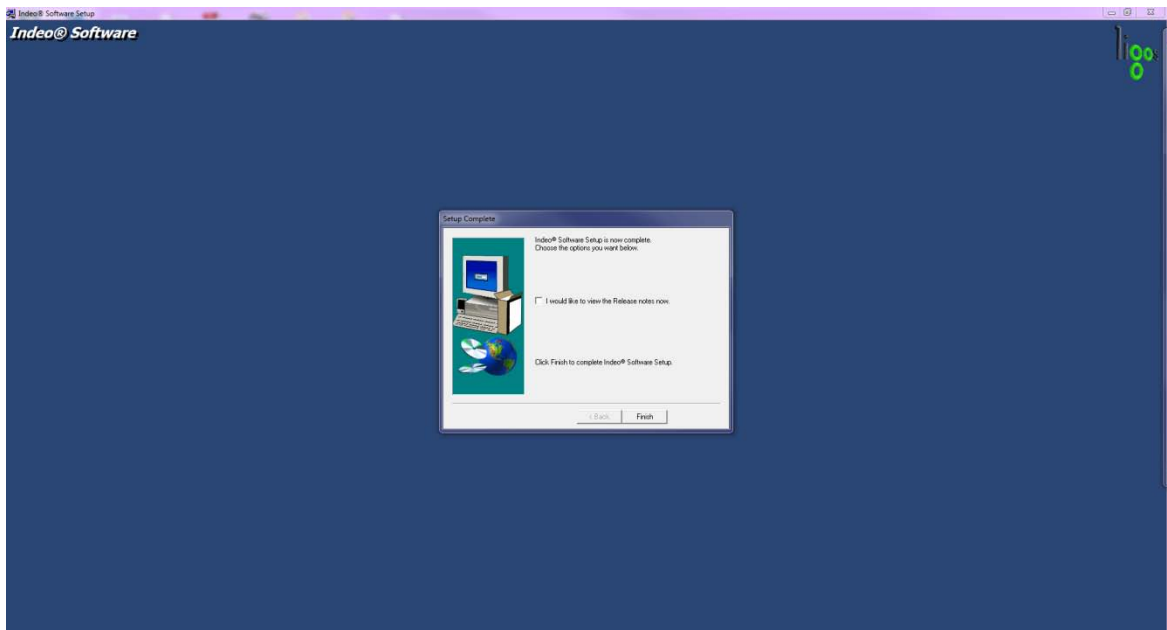


15. **Select Install** to begin installing USB2.0 PC Camera(SN9C201&202)

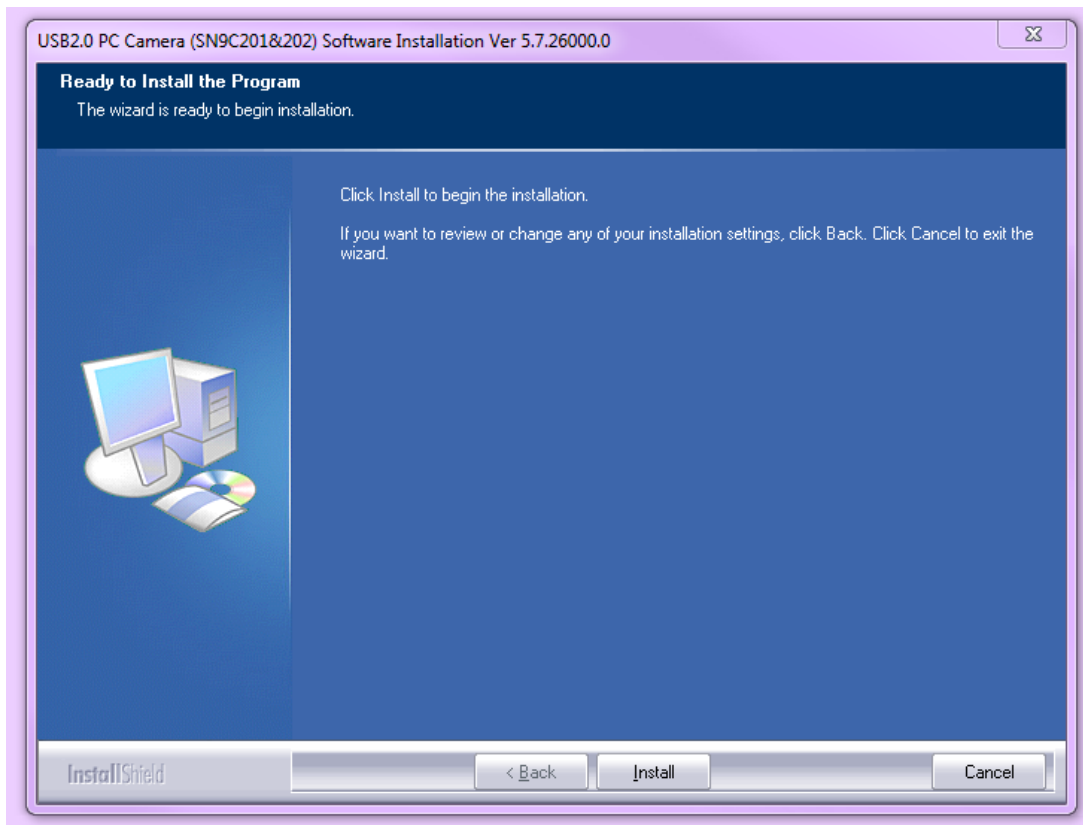


16. **Select Install** to allow device installation.

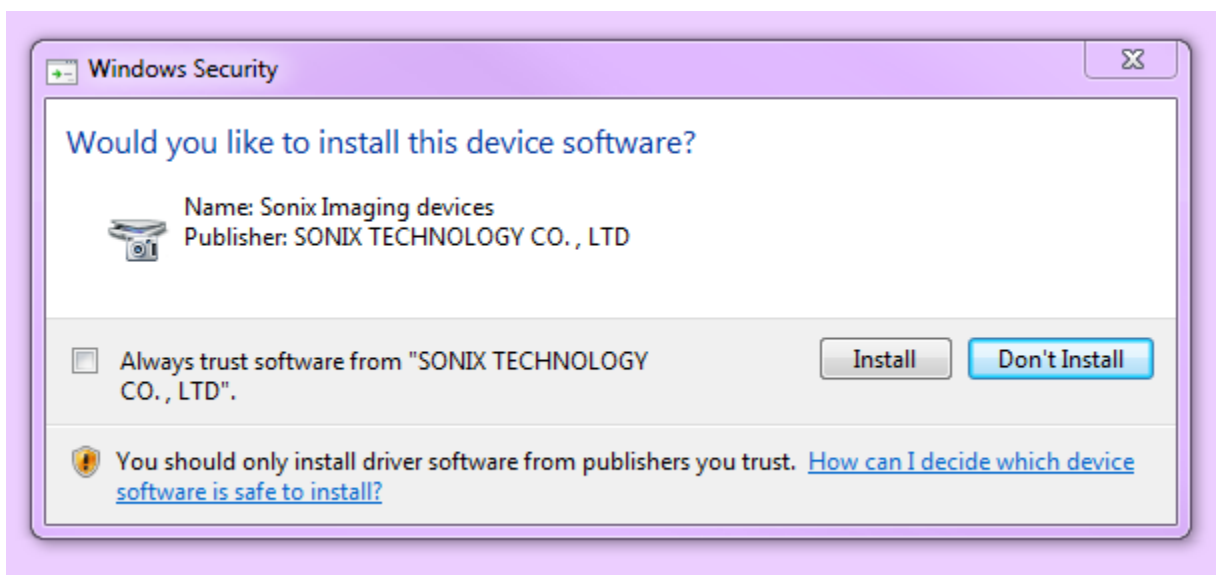


17. **Select Finish** when USB2.0 PC Camera installation is completed.

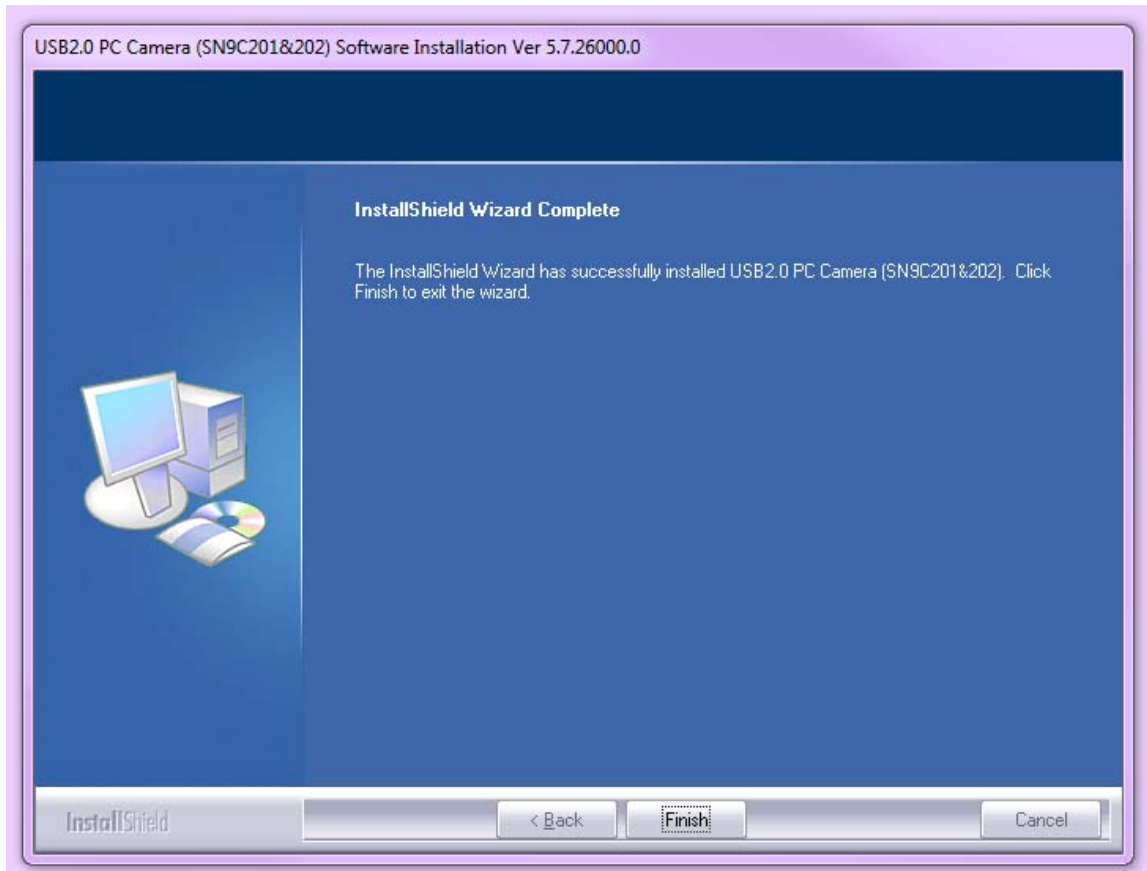

18. Select **Finish** to restart computer.

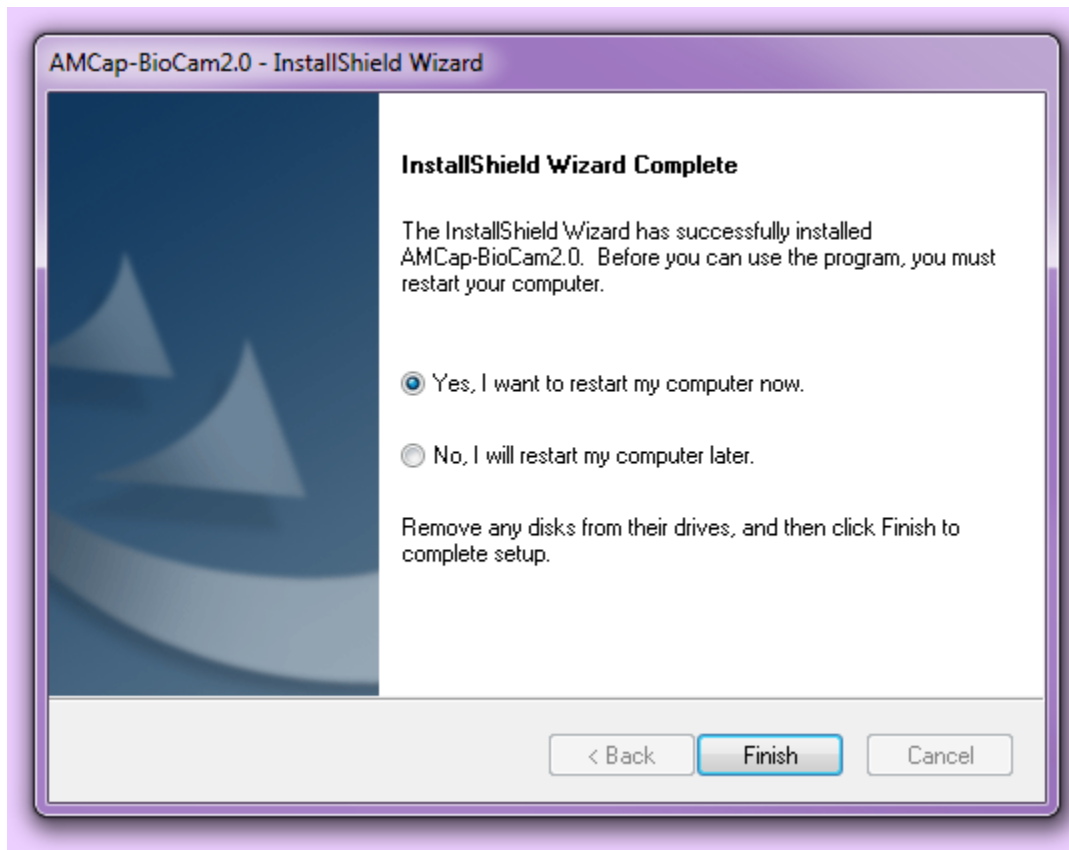


19. Plug in the BioCam2.0 Video otoscope after your system has been rebooted. Your computer will detect and install the device driver software

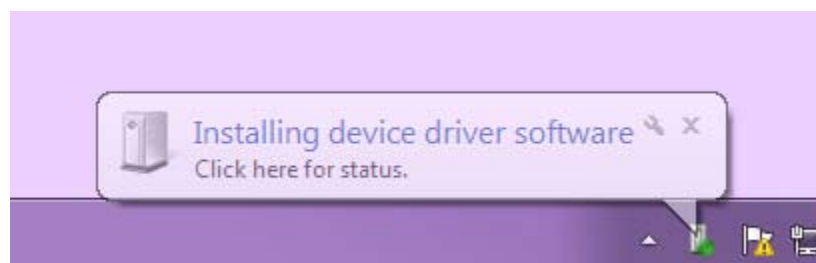


20. Wait for the system to install the driver. Once it is completed, your BioCam2.0™ is ready. **Before you START USING the BioCam2.0 You will have to change the permissions on the BioCam2.0 folder for it to work.**

21. Go to your C Drive. Click on Program Files(x86).

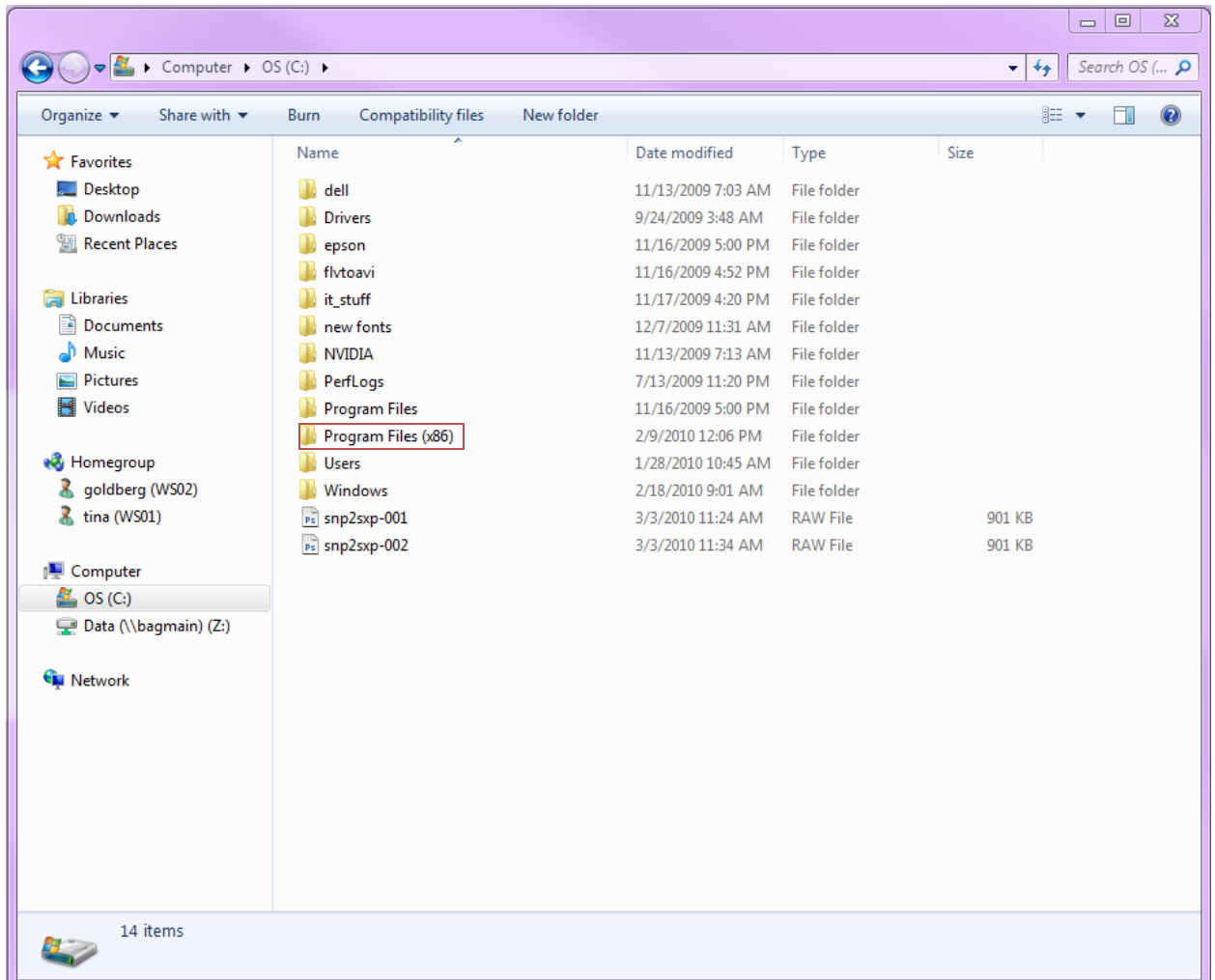





22. Click on the BioCam Folder to access the Capture folder .

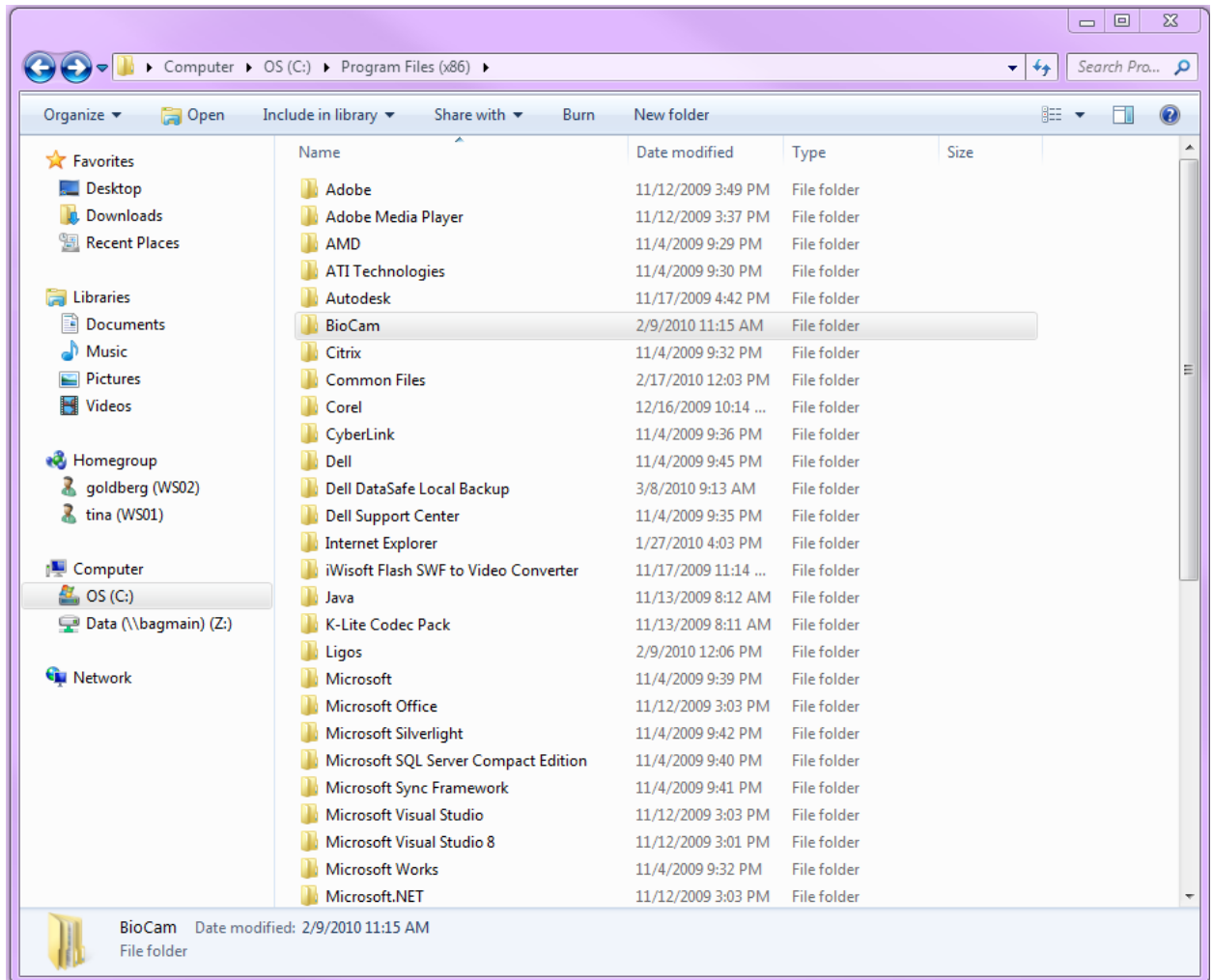

23. Right Click on the capture folder to Change the Security settings for the BioCam2.0 Software.
Click on the Security tab on the Capture properties.

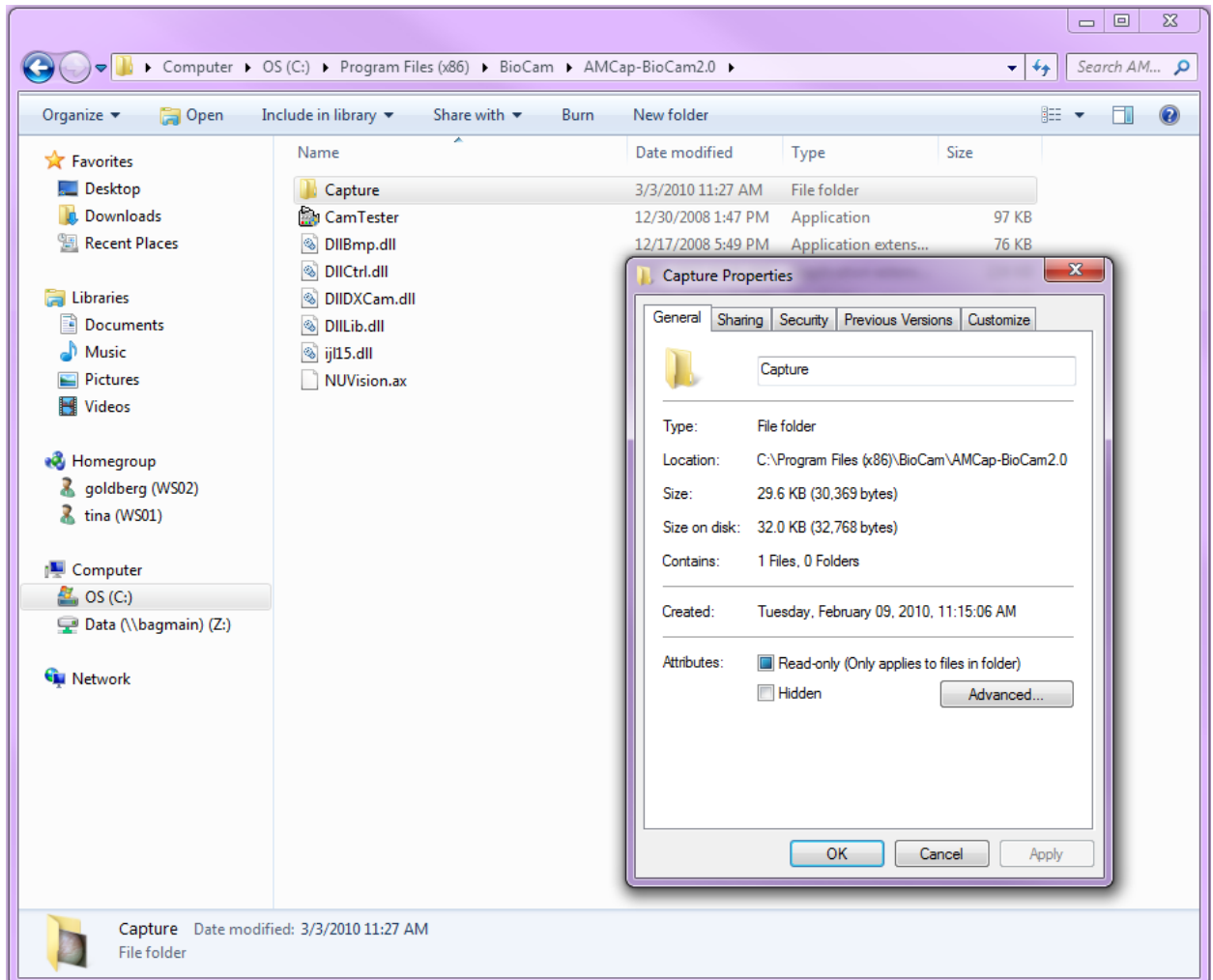

24. Select Users and change permission to allow full control. Click OK and Apply.

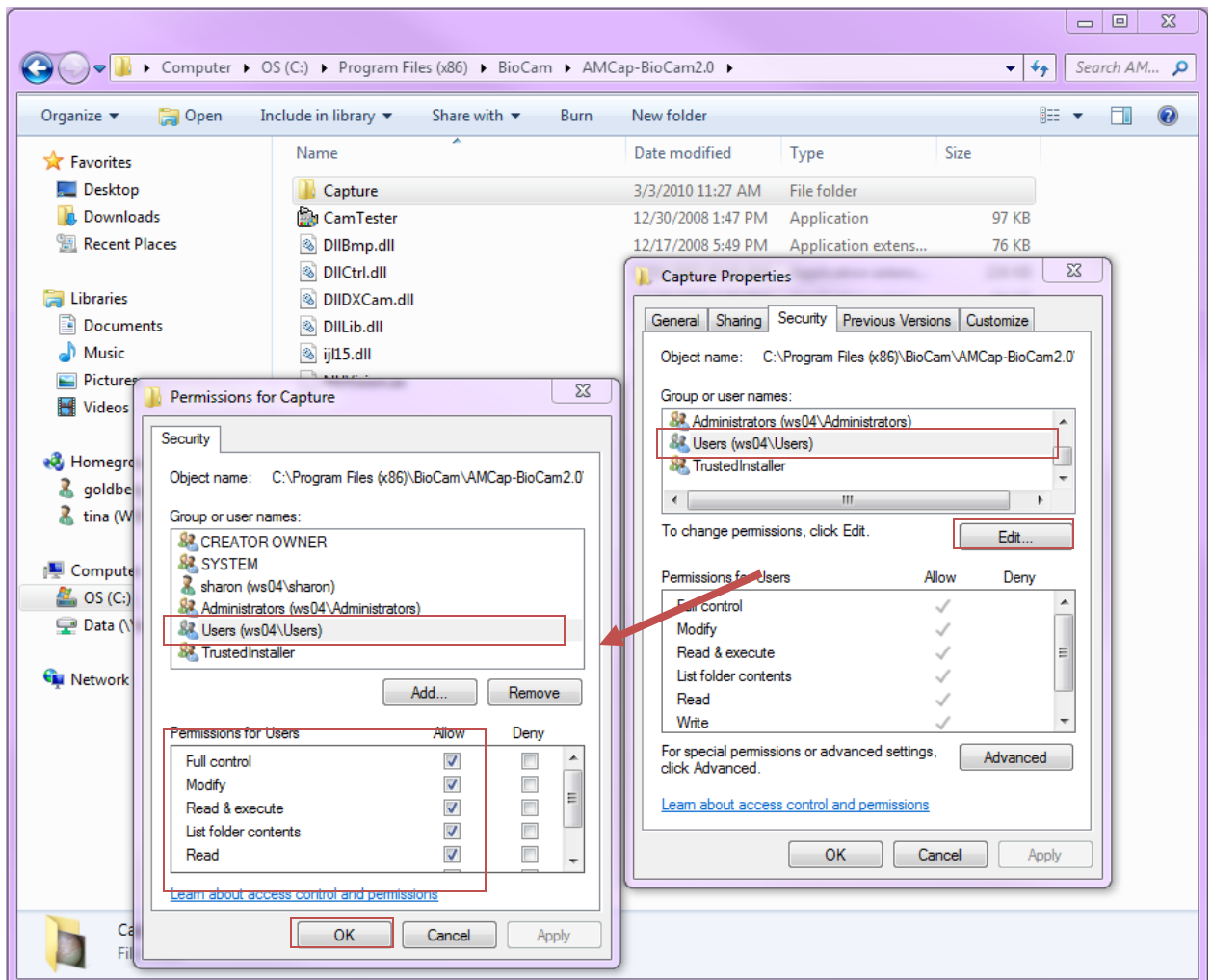

25. Now you are ready to use the BioCam2.0.